

FACT SHEET FOR HEALTHCARE PROVIDERS

Lighthouse Lab Services -
CovidNow SARS-CoV-2 Assay Kit

Updated: November 13, 2020

Coronavirus
Disease 2019
(COVID-19)

This Fact Sheet informs you of the significant known and potential risks and benefits of the emergency use of the CovidNow SARS-CoV-2 Assay Kit.

Testing is to be conducted on specimens from people who meet Coronavirus Disease 2019 (COVID-19) clinical and/or epidemiological criteria for testing.

All patients whose specimens are tested with this assay will receive the Fact Sheet for Patients: Lighthouse Lab Services - CovidNow SARS-CoV-2 Assay Kit.

What are the symptoms of COVID-19?

Many patients with confirmed COVID-19 have developed fever and/or symptoms of acute respiratory illness (e.g., cough, difficulty breathing). The current information available to characterize the spectrum of clinical illness associated with COVID-19 suggests that symptoms include cough, shortness of breath or dyspnea, fever, chills, myalgias, headache, sore throat or new loss of taste or smell. Based on what is known about the virus that causes COVID-19, signs and symptoms may appear any time from 2 to 14 days after exposure to the virus. Based on preliminary data, the median incubation period is approximately 5 days but may range 2-14 days.

Public health officials have identified cases of COVID-19 throughout the world, including in the United States. Please check the CDC webpage for the most up to date information.

What do I need to know about COVID-19 testing?

Current information on COVID-19 for healthcare providers is available at CDC's webpage, *Information for Healthcare Professionals* (see links provided in "Where can I go for updates and more information" section).

- The CovidNow SARS-CoV-2 Assay Kit can be used to test upper and lower respiratory specimens (such as nasopharyngeal or oropharyngeal swabs, bronchoalveolar lavage, sputum, lower respiratory tract aspirate, nasopharyngeal wash/aspirate or nasal aspirate).

This test is to be performed only using upper respiratory specimens collected from individuals who meet COVID-19 clinical and/or epidemiological criteria for testing.

- The CovidNow SARS-CoV-2 Assay Kit should be ordered for the detection of the virus that causes COVID-19 in individuals who meet the COVID-19 clinical and/or epidemiological criteria for testing.
- The CovidNow SARS-CoV-2 Assay Kit is authorized for use in laboratories in the United States, certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA), 42 U.S.C. §263a, to perform high complexity tests

Specimens should be collected with appropriate infection control precautions. Current guidance for COVID-19 infection control precautions are available at the CDC's website (see links provided in "Where can I go for updates and more information" section).

Use appropriate personal protective equipment when collecting and handling specimens from individuals suspected of being infected with COVID-19 as outlined in the CDC *Interim Laboratory Biosafety Guidelines for Handling and Processing Specimens Associated with Coronavirus Disease 2019 (COVID-19)*. For additional information, refer to CDC *Interim Guidelines for Collecting, Handling, and Testing Clinical Specimens from Persons Under Investigation (PUIs) for Coronavirus Disease 2019 (COVID-19)* (see links provided in "Where can I go for updates and more information" section).

What does it mean if the specimen tests positive for the virus that causes COVID-19?

A positive test result for COVID-19 indicates that RNA from SARS-CoV-2 was detected, and the patient is presumptively infected with the virus and presumed to be contagious. Laboratory test results should always be considered in the context of clinical observations and epidemiological data in making a final diagnosis and patient management decisions. Patient management

Report Adverse events, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**

FACT SHEET FOR HEALTHCARE PROVIDERS

Lighthouse Lab Services -
CovidNow SARS-CoV-2 Assay Kit

Updated: November 13, 2020

Coronavirus
Disease 2019
(COVID-19)

decisions should be made with a healthcare provider and follow current CDC guidelines.

The CovidNow SARS-CoV-2 Assay Kit has been designed to minimize the likelihood of false positive test results. In the event of a false positive result, risks to patients could include the following: a recommendation for isolation of the patient, monitoring of household or other close contacts for symptoms, patient isolation that might limit contact with family or friends and may increase contact with other potentially COVID-19 patients, limits in the ability to work, the delayed diagnosis and treatment for the true infection causing the symptoms, unnecessary prescription of a treatment or therapy, or other unintended adverse effects.

All laboratories using this test must follow the standard confirmatory testing and reporting guidelines according to their appropriate public health authorities.

What does it mean if the specimen tests negative for the virus that causes COVID-19?

A negative test result for this test means that SARS-CoV-2 RNA was not present in the specimen above the limit of detection. However, a negative result does not rule out COVID-19 and should not be used as the sole basis for treatment or patient management decisions. A negative result does not exclude the possibility of COVID-19.

When diagnostic testing is negative, the possibility of a false negative result should be considered in the context of a patient's recent exposures and the presence of clinical signs and symptoms consistent with COVID-19. The possibility of a false negative result should especially be considered if the patient's recent exposures or clinical presentation indicate that COVID-19 is likely, and diagnostic tests for other causes of illness (e.g., other respiratory illness) are negative. If COVID-19 is still suspected based on exposure history together with other clinical findings, re-testing should be considered by healthcare providers in consultation with public health authorities.

Risks to a patient of a false negative include: delayed or lack of supportive treatment, lack of monitoring of

infected individuals and their household or other close contacts for symptoms resulting in increased risk of spread of COVID-19 within the community, or other unintended adverse events.

Is this test FDA Approved?

No. This test is not yet approved or cleared by the United States FDA. When there are no FDA-approved or cleared tests available, and other criteria are met, FDA can make tests available under an emergency access mechanism called an Emergency Use Authorization (EUA). FDA has been notified of the distribution of this test, and the EUA for this test is currently under review provided by the Secretary of Health and Human Service's (HHS's) declaration that circumstances exist to justify the emergency use of in vitro diagnostics for the detection and/or diagnosis of COVID-19. This test is being distributed in compliance with FDA's Policy for Coronavirus Disease-2019 Tests During the Public Health Emergency (Revised) (the "Policy").

The United States FDA has declared an emergency access mechanism called an Emergency Use Authorization (EUA). The EUA process is supported by the Secretary of Health and Human Service's (HHS's) declaration that circumstances exist to justify the emergency use of in vitro diagnostics (IVDs) for the detection and/or diagnosis of COVID-19.

An IVD made available under the FDA's Policy or an EUA has not undergone the same type of review as an FDA-approved or cleared IVD. FDA may issue an EUA when certain criteria are met, which includes that there are no adequate, approved, available alternatives, and based on the totality of scientific evidence available, it is reasonable to believe that this IVD may be effective in diagnosing COVID-19.
)

Report Adverse events, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**

FACT SHEET FOR HEALTHCARE PROVIDERS

Lighthouse Lab Services -
CovidNow SARS-CoV-2 Assay Kit

Updated: November 13, 2020

Coronavirus
Disease 2019
(COVID-19)

Where can I go for updates and more information?

CDC webpages:

General: <https://www.cdc.gov/COVID19>

Healthcare Professionals:

<https://www.cdc.gov/coronavirus/2019-nCoV/guidance-hcp.html>

Information for Laboratories: <https://www.cdc.gov/coronavirus/2019-nCoV/guidance-laboratories.html>

Laboratory Biosafety: <https://www.cdc.gov/coronavirus/2019-nCoV/lab-biosafety-guidelines.html>

Isolation Precautions in Healthcare Settings:

<https://www.cdc.gov/coronavirus/2019-ncov/infection-control/control-recommendations.html>

Specimen Collection: <https://www.cdc.gov/coronavirus/2019-nCoV/guidelines-clinical-specimens.html>

Infection Control: <https://www.cdc.gov/coronavirus/2019-ncov/infection-control/index.html>

FDA webpages:

General: www.fda.gov/novelcoronavirus

EUAs:(includes links to patient fact sheet and manufacturer's instructions) <https://www.fda.gov/medical-devices/emergency-situations-medical-devices/emergency-use-authorizations>

Manufacturer: Lighthouse Lab Services

LMSI, LLC d/b/a Lighthouse Lab Services

9731-J Southern Pine Blvd.

Charlotte, NC 28273

support@lighthouselabservices.com

<https://www.lighthouselabservices.com/covidnow-sars-cov-2-assay/>

Report Adverse events, including problems with test performance or results, to MedWatch by submitting the online FDA Form 3500 (<https://www.accessdata.fda.gov/scripts/medwatch/index.cfm?action=reporting.home>) or by calling **1-800-FDA-1088**